

# How can I save a certain camera angle?

When in SRV (Single Room View), PTZ camera(s) assigned to the room can be adjusted to a customized angle.



**IMPORTANT:** The controls of the PTZ function are visible only when hovering over the camera feed:

***Pan and Tilt movement*** - by dragging the target sign where desired or using the respective arrows in the bottom right corner

***Zoom in and out*** - by clicking the magnifier with the plus or minus sign respectively

Once the desired view is set, click the **Preset** button to prompt the **Preset** menu with the **Add new preset** field to open.

Type the name of the preset in the field and click the **plus** button.



**TIP:** Previously saved camera views can be used by clicking their name in the **Preset** menu.

## Related topics

- [Annotations](#)
- [Annotations](#)
- [Cameras](#)
- [Can a video be uploaded to the system?](#)
- [Center Overview \(CO\)](#)
- [Data Entry during Simulation](#)
- [How can annotations be added to a live recording?](#)
- [How can I delete a recording?](#)
- [How can I download video recordings created in LearningSpace?](#)
- [How can I save a certain camera angle?](#)